

HEALTHCARE KIT



CHECK OUT OUR PAO PI BODY-CHECK/FRQ.COM OR CONTACT US BY EMAIL AT CONTACT® BODYCHECKUP.COM



YOUR SLIP IS **VALID FOR 12 MOUTHS.** IF YOU EXCEED THIS DATE, PLEASE **CONTACT US.**

poqycheckup.com

TO SEND YOUR SAMPLES, PLEASE USE THE PREPAID DHL VOUCHER.



WATCH THE VIDEO TO SEE HOW TO SEND YOUR SAMPLES

> CAUTION

- Book an appointment on DHL at least 24H before taking your samples.
- All the information
 concerning samples and how to return

your samples are in the user manual in the kit.

• Do not send

samples after noon on Thursday, during weekends or the last day or two before a public holiday.

 Do not pay anything to DHL transportation is prepaid and already included in the price for your analyses.



> 01 Where can you find your DHL shiping number?

Then, connect to <u>mydhl.express.dhl</u> and select your location

> 02

Once you're on mydhl.express.dhl - Click on the **'Ship'** button, then

- Click on 'Schedule a pickup'



SCAN ME

> 03

When asked, "Do you need to create a shipping label?" - Answer, **"No"**

> 04

Enter your shipping number that appears under the first barcode (without writing WAYBILL) and - Click on **'Next'**

> 05

Fill in the details of the pickup address, then

- Click on 'Next'

- Specify the package information (weight 0.3kg/size: 24x12x5cm)

- Click on 'Next'

> 06

Indicate your time of availability at which you want the driver to pick up the package, then

- Click on 'Schedule Pickup'

